

**OPERATOR'S CHECKLIST FOR ZOLL M / E SERIES
SEMI-AUTOMATIC DEFIBRILLATOR / MONITOR**

Checks to be conducted weekly
(For more detailed information, see the M / E series Operator's guide)

1. Condition

Unit clean, no spills or dust, clear of objects on top, case intact

2. Multifunction Pads

1 set pre-connected/ 1 spare

3. Inspect cables for cracks, broken wires, connector

A ECG electrodes cable, connector

B Multi-function cable, connector

4. Batteries

Fully charged battery in unit

5. Disposable supplies

A Multifunction Pads in **sealed** pouches – 2 sets

B ECG electrodes

C Recorder paper

D Alcohol wipes

6. Operational checks

A Power On sequence

Turn unit ON, 4-beep tone heard

B Defibrillator (In Semi-Automatic mode)

Ensure Multi-function cable connected to test connector. "CHECK PADS" displays.

Press **ANALYSE** button, unit charges to 30 J

Press and hold **SHOCK** button, "TEST OK" message is displayed

(In Manual mode)

Multi-function cable connected to test connector

Set Defib energy to 30 joules

Press **CHARGE**

Press and hold **SHOCK** button, "TEST OK" message is displayed

C Recorder

Press **RECORDER** button; Recorder runs

Press **RECORDER** button again; Recorder stops

Inspect Recorder printing

7. Please consider the following after each use of this checklist

No action required

Minor problem(s) corrected

Disposable supplies replaced

Major problem(s) identified – UNIT OUT OF SERVICE. Arrange repair.

Developed by: Professional Practice Group	Page 1	Reviewed:
Endorsed by:		
Release Date: November 2005		Next Review: Nov 2008